



# CarieScan<sup>™</sup> Sensors

## Directions For Use

### Introduction

In the box are 50 x CarieScan Sensor packs. Each pack contains a Tufted Sensor and a Soft Tissue Contact, both of which are single use and disposable. The Tufted Sensor is used for measuring occlusal and accessible smooth surfaces.



**Tufted Sensor**



**Soft tissue contact**

### Warnings:

A new sensor pack must be used for EACH patient at each new visit.

The sensors are single use items and **MUST NOT** be sterilised pre-use or for re-use. To ensure reliability and safety, only sensors designed and manufactured by CarieScan Ltd can be used.

### Use

Open the blister pack by peeling off the Tyvek lid.

After taking suitable cross-infection precautions hold the CarieScan with the display facing you and push the Tufted Sensor onto the CarieScan PRO with the tuft facing away from you, locating it with a click.

Attach the Soft Tissue Contact to the cable which is, in turn, plugged into the PRO. The contact is then placed over the lip of the patient's mouth.

The CarieScan is ready for use.

When finished, remove the sensor by squeezing the sensor on the textured side grips to release the catch then remove from the CarieScan.

### Note

Please carefully review the CarieScan PRO user manual for the caries detection protocol and further warnings / indications.